

USB Driver Installation Guide

[Installation]

Step1: Execute the auto-unzip file (CustomDriverInstall_20110208.exe).

Step2: Enter the folder named "CustomDriverInstall_20110208".

Step3: Execute the installation file "**USBVCPInstaller.exe**" and follow the instruction accordingly. If the message asks you to restart the computer, please pick a proper occasion to do it as well.

[Compatible OS]

This version is compatible with Windows XP and Windows 7 (x86 and x64 version).

If you would like to use on Windows Vista, please skip the security warning and continue the installation process.

[Un-installation]

Go to the Control Panel -> Programs -> Programs and Features, select the item "USB Medical Device(Driver removal)" and uninstall it.